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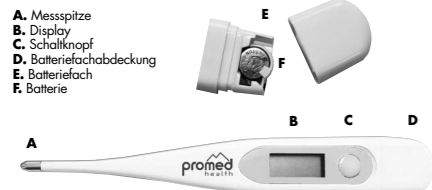
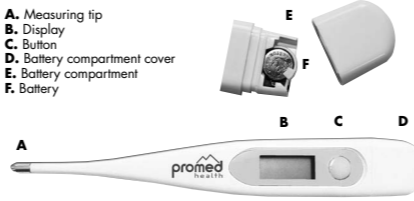
promed EN I Manual digital thermometer PFT-3.7

- A.** Measuring tip
- B.** Display
- C.** Button
- D.** Battery compartment cover
- E.** Battery compartment
- F.** Battery

TECHNICAL DATA

Device type:

Digital thermometer for oral, axillary and rectal measurements

Measuring range:

32,0° C - 42,9° C

Scaling:

0,1° C

Measuring accuracy:

± 0,1° C for body temperature
± 0,2° C for room temperature
(-22° C)

Battery:

1.5 VDC (LR41)

Energy consumption:

0.2 mW

Service life of the battery in continuous operation:

Approx. 200 hours

Dimensions in mm (LxWxH):

133 x 21 x 12

Weight:

10 g

DESCRIPTION AND APPLICATION OF THE DEVICE

Thank you very much for choosing the digital clinical thermometer **Promed PFT-3.7**. In order to ensure the quality and safety of the device, this thermometer was designed and produced in compliance with the medical directive 93/42/EEC. We as the producers cannot be held liable in any way for injury to persons or damage to property arising from the non-observance of this manual.

BEFORE USING THE THERMOMETER FOR THE FIRST TIME

1. Read these instructions of use with care and store them in a safe place.
2. Make sure that the thermometer has not been damaged during transport.

COMMISSIONING

Prior to initial commissioning, please remove the contact protection strip of the battery. For this purpose, please proceed as described in section "Changing the Batteries".

- A.** To start the thermometer, you must press the button **(C)**. The information 188.88mE appears on the display **(B)** and you hear an acoustic signal once. This indicates that the thermometer is operable.
- B.** Once you have released the button **(C)**, the value measured last and the letter "M" appear on the display **(B)** for 2 seconds. Then the values 37.00°C/98.6°F appear. If the ambient temperature is below 32.0°C/90°F, Lo°C/Lo°F is indicated on the display. As soon as the "C"/°F flashes on the display **(B)**, the thermometer is ready for use.
- C.** Once the maximum measuring temperature has been reached, the °C/°F no longer flashes on the display **(B)** and you hear the peeping acoustic tone three times. If the measured temperature exceeds 37.8°C/100°F, you hear an acoustic tone peeping in continuous intervals. This indicates that you have a temperature, and you should read off the measured value.
- D.** Now you can switch off the thermometer by pressing the button **(C)** once. If you do not switch off the thermometer, it will automatically be switched off after 10 minutes.

E. The **Promed PFT-3.7** thermometer can be used for oral, axillary and rectal measurements.

Oral measurement

- Lay the thermometer with the sensor tip onto your tongue and close your mouth. Breathe through your nose during measurement to prevent any falsification of the measuring results. The measuring duration is approx. 1 minute.
- The deviation from the actual body temperature may be -0.4° C to -1.5° C for oral measurement.

Axillary measurement (measurement in the axilla)

- Clean your axilla before measurement. Lay the thermometer with the measuring tip in your axilla. Lower your arm and press it slightly towards your body during measurement.
- The deviation from the actual body temperature may be -0.7° C to -2.0° C for axillary measurement.

Rectal measurement

- Although many patients perceive rectal measurement as unpleasant, it is the measuring method with no deviation to the actual body temperature.
- We recommend applying rice cream, lubricants or the like to the measuring tip before measurement.
- Lie on your side with slightly bent legs for measuring. If you use the thermometer for toddlers, we recommend laying the toddler on its stomach with its legs hanging down in an "E-position".
- Insert the thermometer approx. 2.3mm into the rectum for measurement. Immediately stop if you sense any resistance when inserting the thermometer! Now you must measure until you hear the acoustic signal.
- If the thermometer has been used for rectal measurement once, it should not be used for any other measuring method due to hygienic reasons.

BATTERY CHANGE

If the battery power is too low, the symbol appears on the display **(B)**. For battery change, you must remove the battery cover **(D)** from the battery compartment **(E)** and remove the old battery **(F)**. Now you can insert a new battery (ensure correct polarity) into the battery compartment **(E)** and apply the battery compartment cover **(D)** to the thermometer.

Switching the unit of measurement

If you want to switch the thermometer from Celsius to Fahrenheit, you must press the button **(C)** and keep it pressed for 3 seconds. As soon as °F (Fahrenheit) instead of °C (Celsius) appears at the top left corner on the display **(B)**, you can release the button **(C)**. Now the thermometer will display the measuring results in Fahrenheit. Use the same procedure if you want to switch from Fahrenheit to Celsius.

CLEANING AND DESINFECTION

The **PROMED thermometer PFT-3.7** is water-proof. Thus it is easy to clean and disinfect. Clean the area around the thermometer tip (approx. 5cm from the measuring tip downwards) after each measurement with mild soap solution or with a cloth soaked in mild alcohol solution.

⚠️ SAFETY INSTRUCTIONS

- Never deposit or use the thermometer on hot surfaces, close to open gas flames or hot objects such as stove tops. Otherwise, the housing will melt.
- Use the thermometer only with batteries approved by the manufacturer.
- Use the thermometer indoors only.
- Keep the thermometer in a dry place.
- Do not bend the measuring tip.
- Do not bite the measuring tip during oral measurement.
- Some kind of supervision is required when using the thermometer around children.
- Make sure that children do not play with the thermometer. It includes small parts (battery and battery cover) which may cause suffocation when swallowed.
- For hygienic reasons, the thermometer should not be used for oral or axillary measurements after rectal measurement.
- Use the thermometer only for applications as specified in these instructions for use.
- The thermometer is exclusively intended for measuring human body temperature, as well as for the measuringspots in the human body as specified in the instructions for use.
- Using the thermometer in the vicinity of strong electromagnetic fields, e.g. next to a mobile telephone, may cause malfunctions.
- Please immediately contact your general practitioner in case of conspicuous results.
- This thermometer is not intended for use by persons (including children) with restricted physical, sensorial or mental abilities or lacking experience and/or lacking knowledge, unless they are supervised by a person responsible for their safety or if they were instructed on the use of the thermometer.
- Never allow children to play with the packing material, as there is a danger of suffocation.

METROLOGICAL CHECK (FOR COMMERCIAL USE)

The thermometer was calibrated by the manufacturer for the duration of two years. For commercial use, a metrological check must be made at least every two years. The check is subject to a charge and can be accomplished by the responsible authorities or by authorised maintenance service agencies in compliance with the "Medical Device Operator Ordinance".

DISPOSAL

Electric appliances, accessories and packaging should be recycled in an environmentally friendly manner.

Only for EU countries:

Do not dispose of electric appliances in the household waste! According to the European Directive 2002/96/EC on Waste Electrical and Electronic Equipment and its transposition into national law, electric appliances which are no longer functional must be separately collected and recycled in an environmentally friendly manner. Within the EU, this symbol indicates that this product must not be disposed of in the household waste. Electronic waste contains recyclable materials which should be recycled in order to prevent damage to the environment and/or human health by uncontrolled waste disposal. Therefore, please dispose of old devices in suitable collection systems or return the device for disposal to the shop where you bought it. They will provide for adequate recycling.

Rechargeable batteries:

Do not dispose of rechargeable batteries in the household waste or throw them into fire or water. Rechargeable batteries must be collected, recycled or disposed of in an environmentally friendly manner.

Only for EU countries:

According to directive 91/157/EEC it is mandatory to recycle defective or used batteries. Batteries which can no longer be used may be returned to: Promed GmbH, Lindenweg 11, D-82490 Farchant.

WARRANTY

This thermometer was manufactured with utmost care and checked before leaving the factory. Therefore this thermometer comes with a warranty of 24 months effective from the date of purchase and in accordance with the following terms and conditions.

- In case of verifiable material and/or manufacturing defects which may occur when using the thermometer in accordance with these instructions and which are identified within the warranty period, we will replace all defective parts of the thermometer free of charge within the warranty period including the share of labour

costs for warranty repairs.

- Warranty will expire in case of the incorrect use of the thermometer (e.g. non-observance of the instructions of use), in case of interventions in the thermometer (e.g. opening the thermometer housing), as well as if spare parts are used which are not approved by **Promed**. Wear parts are not included in the warranty.
- The warranty period starts at the day of purchase. The filing of a warranty claim does not affect the warranty period. Warranty claims must be filed within the warranty period. Complaints after the expiration of the warranty period cannot be considered.
- Warranty claims within the scope of these warranty terms can only be filed if the date of purchase can be verified by a sales receipt or the like.
- Subject to technical and optical changes, as well as changes to the equipment!
- The statutory seller's warranty shall not be affected by our warranty terms.
- **Promed** cannot be held liable for any possible translation errors.
- The following information is mandatory for trouble-free processing:
 1. Original sales receipt or seller's stamp with date of purchase
 2. Identified defect
 3. Device designation / type

EXPLANATION OF THE SYMBOLS

Symbol for WARNING! This symbol refers to the safety-relevant chapters in these instructions of use!

The symbol corresponds to symbol 10, table D1, appendix D of standard EN 60601-1:2006 +AC:2010. Instructions for a device, system or accessory part include important safety-related information for the consumer.

Symbol for type BF (body floating) devices:

The symbol corresponds to symbol 20, table D1, appendix D of standard EN 60601-1:2006 +AC:2010. Devices which comply with the requirements of this standard offer an increased degree of protection against electrical shocks, in particular if admissible patient leakage currents and user safety are taken into account.

Observe the symbol for instructions: The symbol corresponds to symbol 10, table D1, appendix D of standard EN 60601-1:2006 +AC:2010. The instructions provided in the instructions of use must be observed.

Symbol for manufacturer and production site:

The symbol corresponds to symbol 5.1.1 of standard EN 15223-1:2012. It indicates the name and address of the manufacturer and the production site.

Symbol for the date of manufacture:

The symbol corresponds to symbol 5.1.3 of standard EN 15223-1:2012. It indicates the date of production.

Symbol for the notified body:

This symbol corresponds to the European Directive 93/42/EEC on Medical Products and is supplemented by the requirements of Directive 2007/47/EC. 0197 Notified body: TÜV Rheinland [0197].

Symbol for the authorised representative in the European Union

The symbol corresponds to symbol 5.1.2 of standard EN 15223-1:2012. It indicates the name of authorised representatives in the European Union.

EC-Directives

CE according to the EC Directive 93/42/EEC in compliance with the EC Directives 2007/47/EC, 2004/108/EC, 2002/95/EC, 1907/2006/EC, 2005/69/EC, 2001/95/EC and 93/68/EEC. Furthermore, the thermometer meets the requirements of the standards DIN EN 12470:3:2000+A1:2009; DIN EN 60601-1:2006+AC:2010; DIN EN 60601-2:2007+AC:2010, EN 60601-1-11:2010. Device classification: class IIa.

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